



Insio C&G IX CIC

Data Sheet

7IX

5IX

3IX

2IX

1IX

CIC

113/50

- 50 dB / 113 dB SPL
(2 ccm coupler)
- 60 dB / 125 dB SPL
(Ear simulator)

118/55

- 55 dB / 118 dB SPL
(2 ccm coupler)
- 66 dB / 128 dB SPL
(Ear simulator)

124/60

- 60 dB / 124 dB SPL
(2 ccm coupler)
- 70 dB / 135 dB SPL
(Ear simulator)

Insio C&G IX CIC | Technical Data


Type	113 / 50		118 / 55		124 / 60	
	2 ccm coupler	Ear simulator	2 ccm coupler	Ear simulator	2 ccm coupler	Ear simulator
Output sound pressure level						
OSPL90 at 1.6 kHz	—	117 dB SPL	—	118 dB SPL	—	127 dB SPL
Maximum OSPL90	113 dB SPL	125 dB SPL	118 dB SPL	128 dB SPL	124 dB SPL	135 dB SPL
HFA OSPL90	109 dB SPL	—	109 dB SPL	—	120 dB SPL	—
Full-on gain						
FOG at 1.6 kHz	—	53 dB	—	57 dB	—	65 dB
Maximum FOG	50 dB	60 dB	55 dB	66 dB	60 dB	70 dB
HFA FOG	45 dB	—	50 dB	—	57 dB	—
Reference test gain	31 dB	42 dB	32 dB	43 dB	43 dB	52 dB
Frequency, noise and directivity						
Frequency range	7IX 5IX 3IX, 2IX, 1IX	100–10000 Hz 100– 8700 Hz 100– 8200 Hz	100–10600 Hz 100– 8800 Hz 100– 8300 Hz	100–9200 Hz 100– 8700 Hz 100–8200 Hz	100– 9900 Hz 100– 8800 Hz 100– 8300 Hz	100– 7000 Hz 100– 7500Hz
Equivalent input noise	16 dB SPL	19 dB SPL	17 dB SPL	19 dB SPL	16 dB SPL	16 dB SPL
Total harmonic distortion at 500 / 800 / 1600 / 3200 Hz	3 / 3 / 2 / 1 %	4 / 7 / 5 / — %	1 / 2 / 1 / 1 %	2 / 2 / 2 / — %	2 / 2 / 1 / 1 %	3 / 6 / 2 / — %
Tinnitus noiser broadband	71 dB SPL	—	77 dB SPL	—	80 dB SPL	—
Latency	< 15 ms		< 15 ms		< 15 ms	
Inductive coil sensitivity						
MASL (1 mA/m) at 1.6 kHz	—	—	—	—	—	—
Full-on HFA-SPLIV (10 mA/m)	—	—	—	—	—	—
HFA SPLITS (left/right)	—	—	—	—	—	—
RSETS (left/right)	—	—	—	—	—	—
HFA SPLIV	—	—	—	—	—	—
Battery						
Battery runtime (without streaming)	up to 34 h		up to 35 h		up to 35 h	
Battery runtime (incl. 5 h streaming)	—		—		—	
Cellphone compatibility						
Microphone mode	0.65 – 0.96 GHz 1.4 – 2.7 GHz		0.65 – 0.96 GHz 1.4 – 2.7 GHz		0.65 – 0.96 GHz 1.4 – 2.7 GHz	
Telecoil mode	—		—		—	

— not applicable


Refer to section “Further information” for additional information on the values.

Insio C&G IX CIC | Basic Data

113/50

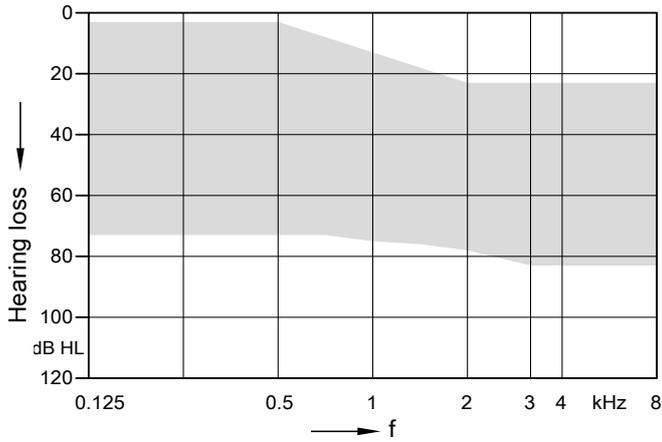


2 ccm coupler

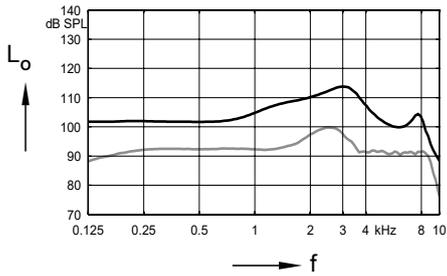


Max. Output sound pressure level
($L_1 = 90$ dB)

Full-on gain
($L_1 = 50$ dB)

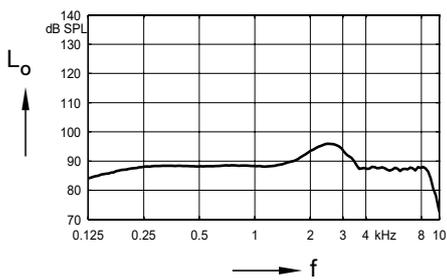


Frequency response
($L_1 = 60$ dB)

Ear simulator

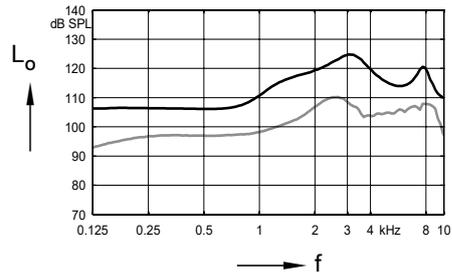

Max. Output sound pressure level
($L_1 = 90$ dB)


Full-on gain
($L_1 = 50$ dB)

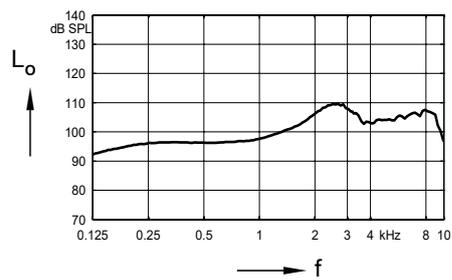


Basic acoustic response
($L_1 = 60$ dB)

Insio C&G IX CIC | Basic Data

118/55

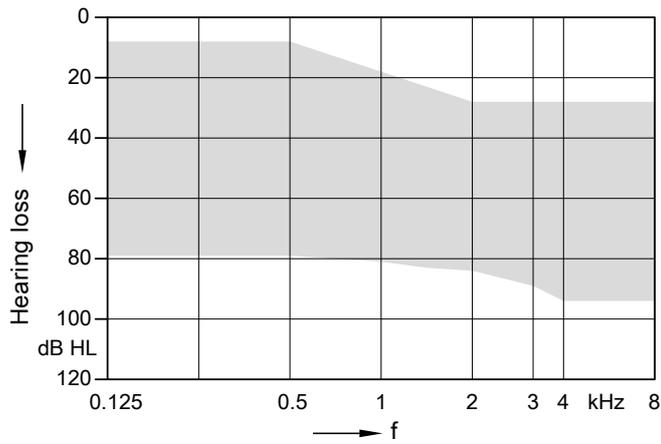


2 ccm coupler

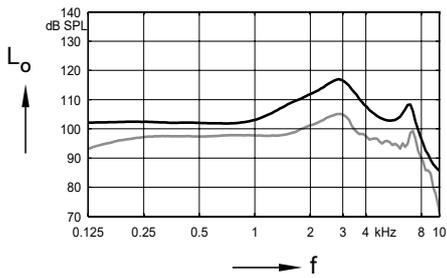


Max. Output sound pressure level
($L_1 = 90$ dB)

Full-on gain
($L_1 = 50$ dB)

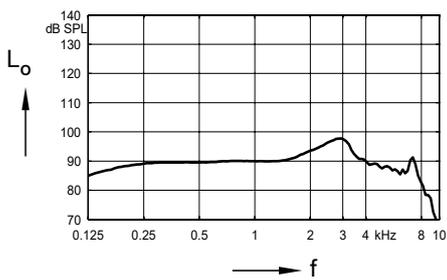


Frequency response
($L_1 = 60$ dB)

Ear simulator

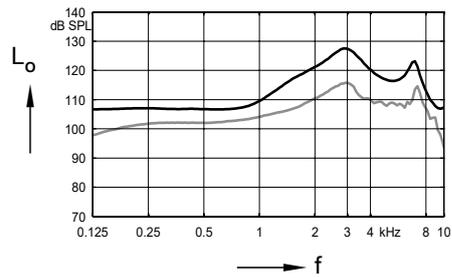

Max. Output sound pressure level
($L_1 = 90$ dB)


Full-on gain
($L_1 = 50$ dB)

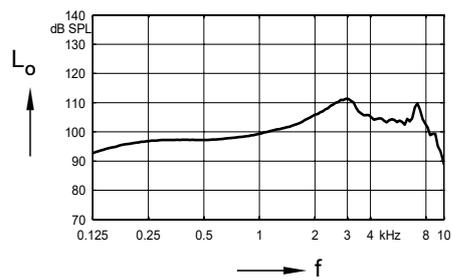


Basic acoustic response
($L_1 = 60$ dB)

Insio C&G IX CIC | Basic Data

124/60

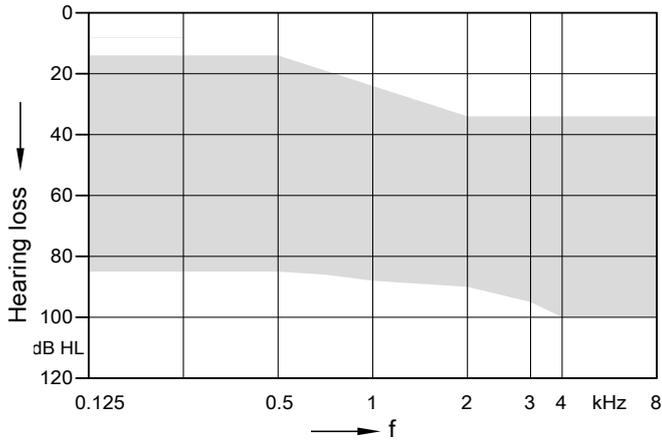


2 ccm coupler

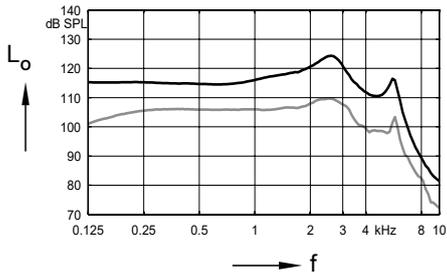


Max. Output sound pressure level
($L_1 = 90$ dB)

Full-on gain
($L_1 = 50$ dB)

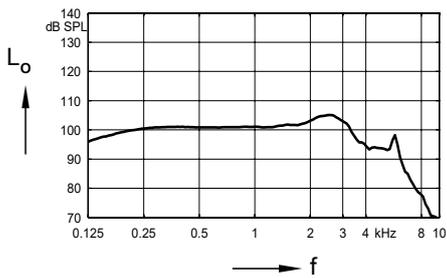


Frequency response
($L_1 = 60$ dB)

Ear simulator

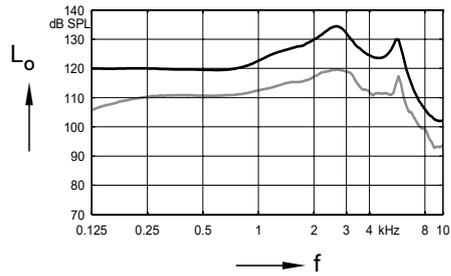

Max. Output sound pressure level
($L_1 = 90$ dB)

Full-on gain
($L_1 = 50$ dB)

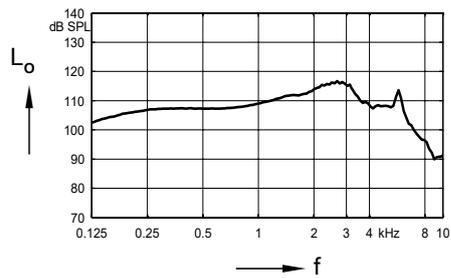

Basic acoustic response
($L_1 = 60$ dB)

Insio C&G IX CIC | Further information

Abbreviations

The following abbreviations are used in this data sheet:

SPL	Sound Pressure Level
OSPL	Output Sound Pressure Level
HFA	High Frequency Average
FOG	Full-On Gain
MASL	Magneto Acoustical Sensitivity Level
SPLITS	Coupler SPL for an Inductive Telephone Simulator
RSETS	Relative Simulated Equivalent Telephone Sensitivity
SPLIV	SPL In a Vertical magnetic field
AI-DI	Articulation Index - Directivity Index
IRIL	Input Related Interference Level
RTF	Reference Test Frequency
ASHA	Audio Streaming for Hearing Aids

Standards and additional information

- All measurements with the 2 ccm coupler were performed according to EN IEC 60118-0:2024 and ANSI S3.22:2014 if applicable.
- All measurements with an ear simulator were performed according to EN 60118-0:1993 + A1:1994 and to DIN 45605 (frequency range) if applicable.
- All Cellphone Compatibility measurements were performed according to EN IEC 60118-13:2020 and ANSI C63.19:2019.
- Cellphone Compatibility definition: It is expected that the hearing aid user can effectively use a compliant wireless device held in a talking position at the ear. Maximum achievable Cellphone Compatibility range: 0.65–0.96 GHz and 1.4–2.7 GHz.
- Curves and figures representing FOG are measured with 20 dB reduction and 70 dB SPL input level.
- Figures representing Equivalent Input Noise incorporate a moderate expansion.
- Tinnitus noiser measurement conditions: all tinnitus single frequency sliders in max position, master volume slider in default position (0 dB) and local volume control in default position.
- Inductive coil sensitivity values, inductive response curves and T ratings apply for instruments with telecoil only.
- The current consumption is measured in reference test setting (RTS) according to the applicable standards. Due to the settling behaviour of hearing aids supporting RF (Radio Frequency), the battery current is measured 3 minutes after turning on (note: no pairing).
- The battery runtime is based on first fit settings using 60 % of the fitting range and an ISTS (International Speech Test Signal) input signal at 65 dB SPL (note: pairing established). The actual battery runtime is determined by battery quality, hearing loss, sound environment, usage and activated feature set. Regarding RF usage, Bluetooth audio streaming from phone to hearing aid and from hearing aid to phone are considered.
- Extended bandwidth up to 12 kHz for 7IX devices only.

Special note for instruments with built-in lithium-ion rechargeable battery

The runtime of all lithium-ion rechargeable batteries reduces over time. The estimates are based on fresh lithium-ion rechargeable battery capacity. Under normal operating conditions, the battery will retain up to 80 % of its initial capacity after 2 years of use. Please note that battery performance will vary depending on individual usage patterns and environmental conditions.

